PIED ROULANT

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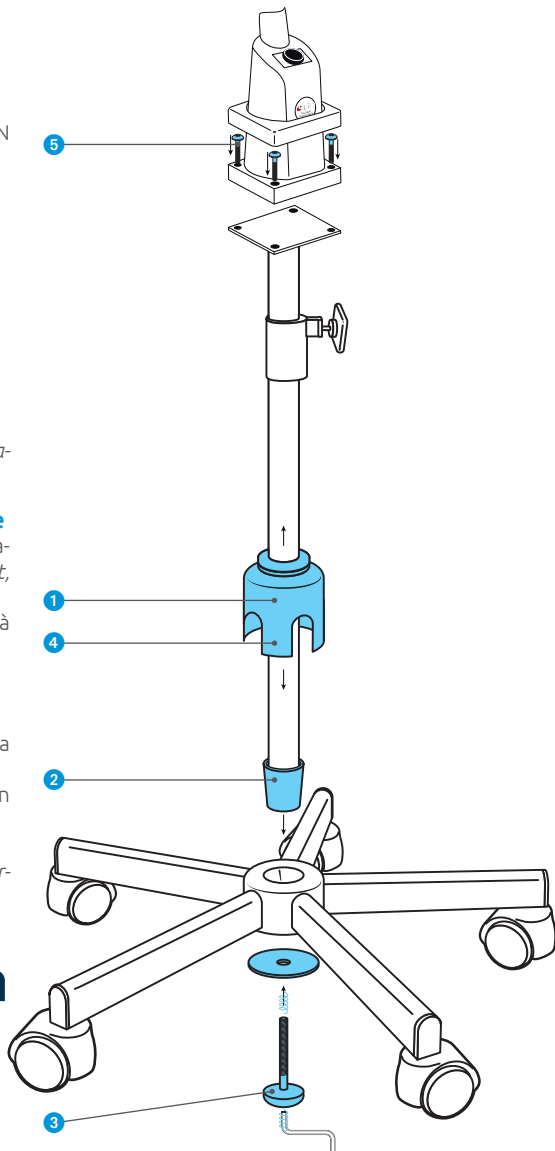
LID adhère à www.ecosystem.eco.

TROLLEY BASIS

Thank you and congratulations for purchasing this trolley base which has been the object of all our care and attention during its manufacture and quality control. This product will give you full satisfaction for many years. Before using it, you should read this notice carefully.

Legal Compliance

CE marking according to Regulation 2017/745, Directives 2014/35/UE, EMC 2014/30/EU and the following harmonized standards: IEC 60.598-1, IEC 60.598-2-25, IEC 60.601-1-2, IEC 60.601-2-41, IEC 60.601-1.

The package contains the following parts

Tube with plastic cap
5 star base + 5 wheels
Plastic cover
Washer
Mounting screws (*base to tube*)
Allen key
User manuel

Uses

This trolley base is specially designed to fit to LID lamps (*with 4 screw*).

Maintenance instructions and cleaning

Mounting the trolley base should be performed by qualified persons.
All cleaning products (*disinfectant, detergent, etc...*) can be used. Difficult stains or spots can be removed with an abrasive cream.

Installation instructions

1. Slide the protective cover along the tube
2. Fit the tube and its plastic cap into the base
3. Tighten the screw with the washer using the provided Allen key
4. Replace the protective cover
5. Fix the lamp on the mounting platform with 4 screws (*provided*)

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